Dr. Mertens
Innovations

Acupressure Octopus®


Lumbosacral Orthosis

Instructions for Use

EN	LUMBOSACRAL ORTHOSIS	3
LV	KRUSTU-JOSTAS ORTOZE	6
LT	KRYŽKAULIO JUOSMENS ORTOZĖ	9
EE	RISTLUUPIIRKONNA ORTOOS	12
RU	ОРТЕЗ ПОЯСНИЧНО-КРЕСТЦОВОГО ОТДЕЛА	15

Acupressure Octopus® Lumbosacral Orthosis
Instructions for Use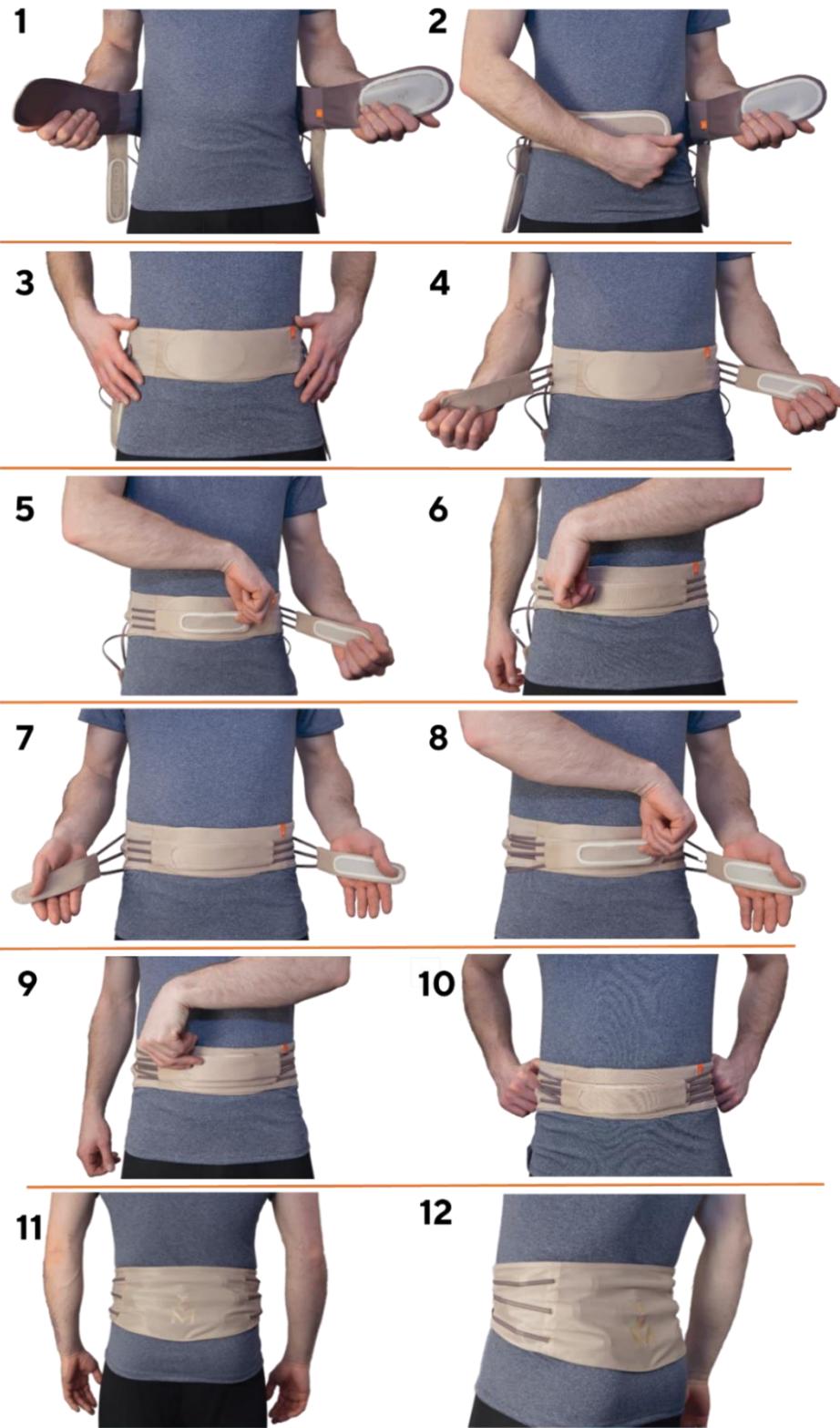

EN

English

1. DEAR CUSTOMER

Thank you for choosing **Dr.Mertens Innovations** product. Our team of professional doctors, engineers, technologists and designers have created an effective products – an Acupressure octopus® orthosis, which relieves tension in the muscles, increases blood microcirculation and relieves back pain based on the natural mobility of the back.

2. INTENDED PURPOSE

Acupressure Octopus® Lumbosacral Orthosis – is medical product. The Acupressure Octopus® Lumbosacral Orthosis model provides support for the back muscles, targeting discomfort in the lumbosacral spine region.

Before using the product, read this manual carefully. Following the instructions included in the manual will provide product's safety and effectiveness. If you have any questions, please consult your care professional or manufacturer.

3. INDICATIONS AND DESIGNATIONS

- Relief of general pain in the lumbosacral spine (acute and chronic);
- Relaxation and body recovery after physical exertion;
- Increase of the movement amplitude;
- Recovery from injuries and post-surgery;
- Massage and pain-relieving effect on the lower back for preventive purposes;
- The benefits of this product are felt particularly during physical activity except contact sports.

4. CONTRAINDICATIONS

ATTENTION! The Acupressure Octopus must NOT be used:

- in severe spinal injuries during the acute period or infections of the spine;
- oncological diseases of the spine;
- in case of skin inflammation, abrasions, wounds in the Acupressure Octopus using area;
- in case of acute abdominal pain;
- during pregnancy;
- chronic obstructive pulmonary disease (COPD).
- by children and people with mental disorders.

5. SAFETY RULES


ATTENTION!

- The product should be used according the manual's recommendations. If you have any questions, please consult your care professional or manufacturer.
- Before first using, check the size of the product. Only good size guarantees safety and effectiveness. A wrong size of the product decreases effect of treatment, increases discomfort and wrong position of the user.
- Pain is always the sign of warning. If the pain maintains or increases, it is advisable to consult your doctor;
- Manufacturer is responsible only for hidden defects in the product or mechanical damages occurred in the natural way of using;
- Do not use the product against its purpose of use;
- Do not use this product in contact sports or in combination with other devices.
- For single patient use only.
- The skin in the area where the product is applied should be clean. Control the skin condition and in case of skin lesions occurring, it is necessary to contact a doctor.
- Avoid contact with ointments, lotions, or substances containing grease or acids.
- When carrying and storing the product, avoid contact with open flames, hot objects and aggressive environments.

6. SIZE CHART


Acupressure Octopus® Lumbosacral Orthosis is available in 7 different sizes. Measure your waist circumference in centimeters and use this table to determine the required orthosis size:

Size	A
(1) XS	59-73 cm
(2) S	67-81 cm
(3) M	75-89 cm
(4) L	83-97 cm
(5) XL	93-109 cm
(6) XXL	105-121 cm
(7) XXXL	117-133 cm

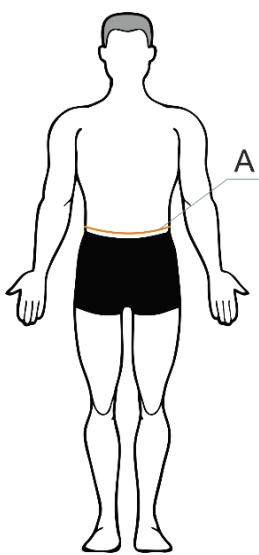When starting to use the product for the first time, we recommend not to overstretch the mechanism, but increase it gradually, find the most suitable stretch within two weeks.

The tightening mechanism of the Acupressure Octopus allows you to choose an individual and pleasant tension, thus controlling the interaction between the Acupressure Octopus and body.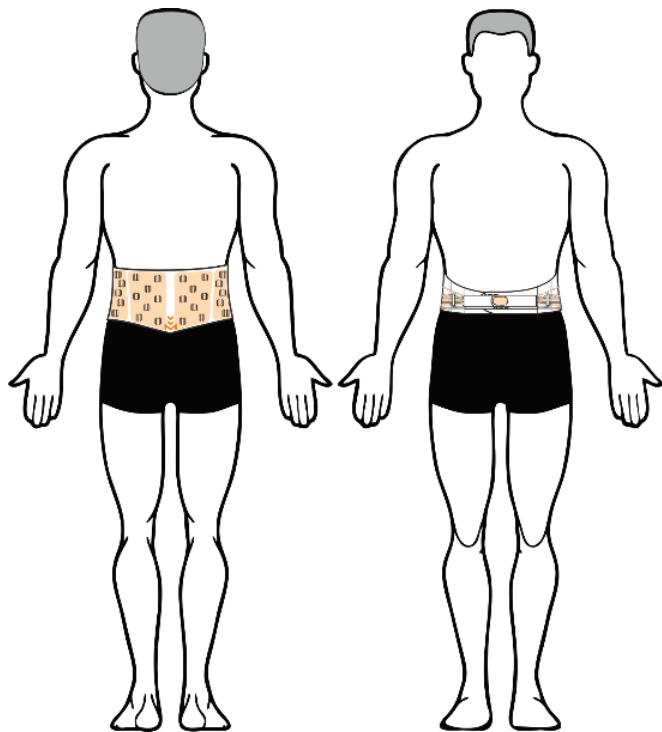

7. NOTES ON HOW TO WEAR THE ORTHOSIS

- The Acupressure Octopus is meant for lumbosacral use;
- The materials used in this product are not harmful to the human body, they do not emit harmful substances. However, for hygienic purposes, it is recommended to wear Acupressure Octopus® over a thin T-shirt;
- Before putting on the Acupressure Octopus, make sure which side is inside, taking into account that the central logo is on the outside (Picture 11);
- The side tightening mechanism is located on the outside of the Acupressure Octopus (Picture 12);

The manufacturer recommends using the Acupressure Octopus for no more than two (2) hours of continuous wear. The Acupressure Octopus may be worn several times a day with an interval of at least two (2) hours.

If the use of the Acupressure Octopus does not cause discomfort, the user may extend the period of continuous use in compliance with individual indications.

Acupressure elements can cause mild local redness on the skin due to pressure (redness disappears within 15 to 45 minutes).

8. APPLICATION INSTRUCTIONS

8.1. INSTRUCTIONS ON HOW TO PUT THE ORTHOSIS ON

Before putting on the Acupressure Octopus, make sure that the direction of putting on the Acupressure Octopus is correct. When putting the Acupressure Octopus in the upright position, make sure that your posture is straight and not tilted forwards. In order to improve the posture, before putting it on we recommend that you take a deep breath and pull your shoulders backwards, bringing the shoulder blades closer together. Changing the position of the head towards the back will give the necessary spinal curvature or lordosis in the lumbar region.

The steps how to put in on are shown visually on Page 2.

When your posture is right:

1. Grasp both ends of the Acupressure Octopus (Picture 1);
2. Tighten with your desired force and close the orthosis (Picture 2 and Picture 3).

In order to be sure that the Acupressure Octopus is worn symmetrically, there is an orange tactile logo tag in the center of the back of the Acupressure Octopus. When pressed, the pressure should coincide with the midline of the spine. In case the Acupressure Octopus is revolved, its position must be adjusted until the position is correct and the Acupressure Octopus is fitted symmetrically. A shifted Acupressure Octopus to one side or the other can cause an asymmetrical effect on the muscles.

When the base of the Acupressure Octopus is put on:


3. Start tensioning the side bands by pulling the closest bands towards the center of the abdomen and connecting them with self-adhesive fasteners (picture 4, picture 5 and picture 6). Lateral retainers provide side muscle massage;
4. If the tension of the side latches is satisfactory, repeat the same tensioning process with further bands (Figure 7, Figure 8 and Figure 9). This fixation will provide massage of the central part of the spine or paravertebral muscles;
5. Make sure that when applying the Acupressure Octopus, it has not slipped down or is not too high (Figure 10 and Figure 11);
6. Bend towards the front, back, right and left sides, thus making sure that the edges of the corset do not press into the rib cage or the gut bone and/or the acupressure elements of the corset do not cause pain (Figure 12).

8.2. INSTRUCTIONS FOR REMOVING THE ORTHOSIS DEVICE

To remove the orthosis, perform the steps described in point 8.1. in reverse order, carefully separating the fasteners from each other. When the orthosis is removed, you can connect the fasteners together and put the product in a storage bag to protect the product from external influences.

9. CARE INSTRUCTIONS

- Wash at a temperature of 30 degrees;
- Use the slow spin mode;
- Do not iron;
- Do not bleach;
- Do not steam;
- Do not wring;
- Drying in a tumble dryer is not recommended.

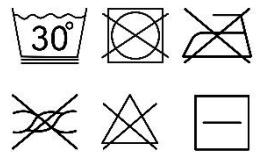IMPORTANT!!!

- Acupressure Octopuses can be washed in a washing machine, by first connecting the self-adhesive fasteners and placing into a washing bag.
- RINSE AID, GREASE, OIL, LOTION, ointment AND SOAP RESIDUE CAN damage the MATERIAL AND CAUSE SKIN IRRITATION.

10. STORAGE INSTRUCTIONS

Store in a dry place, protect from direct sunlight.

11. MATERIAL

SPANDEX
Polyamide
Polyethylen
Stainless steel
Polyester

12. RESPONSIBILITY

The manufacturer accepts no liability if the product is used for other purposes or in a manner which is not in compliance with the instructions for use. Follow the safety instructions and regulations provided in this manual.

13. DISPOSAL

This product can be disposed of together with household waste.

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